

Disinfection

hup DES PLUS XL roll

Instant disinfectant wipes without alcohol



Product name

hup DES PLUS XL roll

Product range

Disinfection

Product description

Quick disinfection wipes without alcohol for cleaning and disinfecting delicate surfaces. Ideally suited for use with the specially designed hup BOX.

Packaging unit

90 towels / pack

Art.-No. 4450304200

Field of application | use

Biocidal Products Regulation: Multi-purpose – can be used in catering, health-care, industry, public buildings, etc. Ideal for disinfecting sports equipment in fitness centres and for disinfecting work areas in hospitals and care facilities. Do not use in food-related areas.

EU Medical Device Regulation: Disinfection of non-invasive medical devices by professionally trained personnel.

Application: Use the hup DES PLUS wipes to fully wet the surfaces and allow to take effect. No rinsing with water required.

UDI-DI

00: UDI-DI 09008353219798

Activity spectrum

VAH/ÖGHMP	-	1 min. *
Surface disinfection with mechanical action: bactericidal, yeasticidal	EN 13624, EN 13727, EN 16615	1 min. *
Limited virucidal effect	EN 14476	30 sec. *
Polyomavirus SV40	EN 14476	2 min. *
Norovirus	EN 14476	30 min. *
Rotavirus	EN 14476	30 sec. **
Limited virucidal effect	EN 16777	1 min. *

* Dirty conditions, medical area. ** Medical area.

All tests performed at room temperature.





Disinfection

hup DES PLUS XL roll [MP]

Instant disinfectant wipes without alcohol

Material compatibility

hup DES PLUS XL roll has been tested for compatibility with the following materials: POM (Hostaform), PP (Moplen), PP (Novolen), Plexiglas, PVC, PE-HD (Hostalen), PA, polysulfone, linoleum flooring, rubber flooring, vinyl flooring, steel (stainless steel). When used on aluminium and non-ferrous metals, material compatibility should first be tested on an inconspicuous spot. Applying the solution to polycarbonate, ABS (Novodur) is not recommended. Minor colour changes may occur on synthetic leather.

Dimensions

H x W: 30.5 x 31 cm

Safety information

Please follow the standard safety measures when handling chemicals. **Use disinfectant safely. Always read the product label before use.** Keep out of reach of children.

Storage

To maintain product quality, store in a cool, dry place.

Product recommendation

To disinfect alcohol-resistant surfaces, we recommend hup DES FORTE wipes.

BAuA Reg.-Nr.

N-108718

Selling unit

Package

Euro pallet

108 packages

Rev. No. 01 / 10.09.2024