

Disinfection

hygienicDES FORTE

Alcohol-based surface disinfectant



Product name

hygienicDES FORTE

Product range

Disinfection

Product description

Ready-to-use, alcohol-based, fast-acting disinfectant for all alcohol-resistant surfaces. Very good material compatibility and skin tolerability. VAH/ÖGHMP listed. Perfume-free.

Packaging unit

6 x 0.9 kg

Art.-No. 4460100808

pH value

neutral


Field of application | use

Ready-to-use, alcohol-based, fast-acting disinfectant for all alcohol-resistant surfaces. Very good material compatibility and skin tolerability. VAH/ÖGHMP listed. Perfume-free. For rapid disinfection also during the working process such as in residential care homes and doctors' surgeries.

Application: Apply undiluted hygienicDES FORTE to the surface, spread with a cloth and let it take effect. Make sure that the surface is fully covered and stays moist the whole time. All visible contaminations must have been removed before disinfection. No rinsing with water required. Local use only.

Activity spectrum

Surface disinfection with mechanical action: bactericidal, yeasticidal	VAH/ÖGHMP; EN 13624, EN 13727, EN 16615	1 min. *
Limited virucidal	EN 14476	1 min. *
Limited virucidal PLUS	EN 14476	3 min. **
Adenovirus	EN 14476	3 min. **
Norovirus	EN 14476	1 min. **
Rotavirus	EN 14476	30 sec. **
Mycobactericidal	EN 14348	1 min. **
Limited virucidal	EN 16777	1 min. *
Limited virucidal PLUS	EN 16777	5 min. **

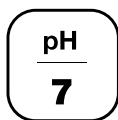



Disinfection

hygienicDES FORTE

Alcohol-based surface disinfectant

Bactericidal/yeastocidal	EN 1276, EN 1650, EN 13697	1 min. *
--------------------------	----------------------------	----------

* Dirty conditions, medical area. ** Clean conditions, medical area.

All tests performed at room temperature.

Material compatibility

Applying the solution onto perspex (PMMA) is not recommended.

Safety information

Please follow the standard safety measures when handling chemicals. **Use disinfectant safely. Always read the product label before use.** Keep out of reach of children. Keep container tightly closed. Keep away from sources of ignition – no smoking.

Storage

To maintain product quality, store closed and in a dry place at room temperature.

Product recommendation

We recommend hygienicDES PLUS for surfaces sensitive to alcohol.

BAuA Reg.-Nr.

N-56133

Selling unit

Piece

Euro pallet

342 pieces