

Disinfection

hygienicDES PLUS


Product name

hygienicDES PLUS

Product range

Disinfection

Product description

Ready-to-use, alcohol-free preparation for cleaning and disinfecting sensitive surfaces. Particularly suitable for surfaces sensitive to alcohol, such as perspex. Leaves a subtle orange scent. Free from alcohols, aldehydes and phenols. ÖGHMP and VAH listed.

Packaging unit

6 x 1 kg system bottle

Art.-No. 4460101808

pH value

neutral

Field of application | use

Biocidal Products Regulation: Alcohol-free surface disinfectant; ideal for cleaning and disinfecting surfaces with direct and frequent skin contact (sunbeds, steam baths, massage tables) as well as gym equipment.

EU Medical Device Regulation: Cleaning and surface disinfection of non-invasive medical devices. This includes areas the patient comes into skin contact with in hospitals and practices, such as examination beds. However, it also includes OP lamps, handrails and equipment control elements after treatment. Applied by professionally trained personnel.

Application: Apply the product to the horizontal surface to be disinfected directly or with the reusable foam or spray gun and spread with a microfibre cloth. When disinfecting vertical surfaces or keyboards, first apply the product to a cloth and then spread it.

UDI-DI

00: UDI-DI 09008353219781

14: UDI-DI 09008353230496

55: UDI-DI 09008353219903

73: UDI-DI 09008353230502

Activity spectrum

VAH/ÖGHMP	-	1 min. *
-----------	---	----------





Disinfection

hygienicDES PLUS

Surface disinfection with mechanical action: bactericidal, yeasticidal	EN 13624, EN 13727, EN 16615	1 min. *
Limited virucidal effect	EN 14476	30 sec. *
Polyomavirus SV40	EN 14476	2 min. *
Norovirus	EN 14476	30 min. *
Rotavirus	EN 14476	30 sec. **
Limited virucidal effect	EN 16777	1 min. *

* Dirty conditions, medical area. ** Medical area.

All tests performed at room temperature.

Material compatibility

hygienicDES PLUS has been tested for compatibility with the following materials: POM (Hostaform), PP (Moplen), PP (Novolen), Plexiglas, PVC, PE-HD (Hostalen), PA, polysulfone, linoleum flooring, rubber flooring, vinyl flooring, steel (stainless steel). When used on aluminium and non-ferrous metals, material compatibility should first be tested on an inconspicuous spot. Applying the solution to polycarbonate, ABS (Novodur) is not recommended. Minor colour changes may occur on synthetic leather.

Safety information

Please follow the standard safety measures when handling chemicals. **Use disinfectant safely. Always read the product label before use.** Keep out of reach of children. Only use visually perfect goods.

Storage

To maintain product quality, store closed and in a dry place at room temperature.

Product recommendation

We recommend hygienicDES FORTE for use on displays.

BAuA Reg.-Nr.

N-108718

Disinfection

hygienicDES PLUS

Selling unit

Piece

Euro pallet

342 pieces

Rev. No. 05 / 10.06.2025